# Contax

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## English

### Product description

Contax is a light-curing, self-etching, and self-conditioning adhesive system for dentine and enamel. The Contax adhesive system can be used with and is compatible with light-, dual- and self-curing composites, compomers and ormocers. If the restoration is not light-cured in the later course of treatment, the activator must be used. Contax itself, however, must be light-cured, even if the Activator is used.

### Indications

Adhesive system for dental composites, compomers and ormocers to be used on dentine and enamel.

### Contra-indications

Do not use the material in the case of

- allergies to components of the materials used, especially benzoyl peroxide if the activator is used,
- exposed pulp,
- pulpitis,
- insufficient oral hygiene.

### Basic Safety Instructions

- ▶ For dental use only!
- ▶ Keep out of reach of children!
- ▶ Avoid contact with the skin! In the event of inadvertent skin contact, wash the affected area thoroughly with soap and water.
- ▶ Avoid contact with eyes! In the event of inadvertent contact with eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.

### Side effects

In isolated cases, contact allergies with products containing similar compositions of ingredients have been recorded and cannot be excluded.

### Interactions with other materials

Do not use together with substances which inhibit polymerization. This is especially valid for products which contain eugenol, such as temporary cements or filling materials.

### Instructions for use

- ▶ Make sure that the primed and/or bonded surface does not become re-contaminated.
- ▶ Do not remove the oxygen inhibition layer caused by the application because this is important for the bonding process with the material applied on top.
- ▶ In deep cavities, material excesses can form at the margin between tooth and matrix ("pooling effect"). Therefore, if possible, place a matrix band only after application of the Bond.
- ▶ Do not interchange lids of bottles because this can lead to a cross-contamination of the liquids. For better differentiation, the lids are color-coded: Red = Primer; Black = Bond; White = Activator.
- ▶ In order to prevent premature polymerization due to ambient light, the materials must be added to the mixing palette (e.g. LuxaTray) immediately prior to application.
- ▶ The Contax-Activator must be used if self- or dual-curing restoration materials are used. Use of the Contax-Activator is not necessary if light-curing restoration materials are used. Contax must be light-cured in all cases.
- ▶ Light-curing units should have an output of 450 nm and should be checked regularly. The light intensity should be a minimum of 400 mW/cm<sup>2</sup>. Place the light as close as possible to the material.
- ▶ DMG recommends the use of a rubber dam.

### Recommended Use

1. Prepare the cavity in accordance with the general rules for adhesive procedures. Bevel the enamel edges. Protect pulp in deep cavities with suitable materials.

**Note:** If additional enamel etching is desired, this can be performed according to manufacturer's instructions. Bond strength of Contax stays the same.

2. Put one drop of Contax-Primer onto the mixing pad or directly onto the single-use brush and work into the tooth structure for 20 seconds.

**Note:** After the application of the Contax-Primer, the surface to be treated must appear evenly moist. If this is not the case, apply Contax-Primer again. It is not necessary to blow on the Contax-Primer.

3. Prepare the bond as follows:

- ▶ For self- or dual-curing restoration materials: Mix one drop of Contax-Bond and one drop of Contax-Activator in a hollow in the mixing pad for approximately 5 seconds.

- ▶ For light-curing restoration materials: Put one drop of Contax-Bond on the mixing pad or directly onto a new single-use brush. Contax-Activator is not necessary.
- 4. Apply the bond or bond-activator mixture to the whole preparation.
- 5. Work the bond or bond-activator mixture into the primed tooth structure for 20 seconds.
- 6. Remove excess.
- 7. Thin out the bond or bond-activator mixture briefly with oil-free air until the surface appears evenly moist.
- 8. Light-cure Contax for 20 seconds.
- 9. Apply the restoration material according to the manufacturer's instructions.

#### Storage and shelf life

- ▶ Store in a dry place between 2–25 °C.
- ▶ Close bottles immediately after use.
- ▶ Do not use the material after the expiration date or if the liquid shows a jelly-like consistency.

#### Composition

Contax-Primer: Water, carboxylic acid, sodium fluoride

Contax-Bond: Hydrophilic and acidic Bis-GMA-based resin matrix, catalyst

Contax-Activator: Hydrophilic and acidic Bis-GMA-based resin matrix, benzoyl peroxide

#### Packaging

Intro Kit	
Intro Kit 1 bottle @ 5 ml Contax-Primer, 1 bottle @ 5 ml Contax-Bond, 25 Single-use brushes White, 25 Single-use brushes Red, 1 mixing pad	REF 220263
Intro Kit + Activator 1 bottle @ 5 ml Contax-Primer, 1 bottle @ 5 ml Contax-Bond, 1 bottle @ 5 ml Contax-Activator, 25 Single-use brushes White, 25 Single-use brushes Red, 1 mixing pad	REF 220282
Refill	
1 bottle @ 5 ml Contax-Primer	REF 220281
1 bottle @ 5 ml Contax-Bond	REF 220284
1 bottle @ 5 ml Contax-Activator	REF 220283
Accessories	
25 Single-use brushes White	REF 109320

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