O-Bite Scan

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Vinylpolysiloxane, Hydrogenpolysiloxane, Füllstoffe, Pigmente, Additive, Platinkatalysator

Handelsformen

2 Kartuschen á 50 ml Paste, 12 Automix-Tips	REF 919017
8 Kartuschen á 50 ml Paste, 50 Automix-Tips	REF 919018

Instructions for use

English

Product description

Scannable bite registration material based on VPS with increased final hardness. The material is suitable for all scanners designed for scanning impression materials (e.g. the D700 dental scanner from 3Shape). Additional surface treatment is not required.

Indications

- Bite registration
- Material for application on bite forks for use with facebows
- Key material, e.g. for intraoral incisal pin registration

Contraindications

- ▶ Do not use the material in cases of allergies to any of the ingredients or, as can occur in rare cases, in the event of contact allergies.

Basic safety information

- ▶ For dental use only!
- ▶ Keep out of the reach of children!

Interactions

- ▶ Do not allow the material to come into contact with organic solvents or liquids that contain solvents as this can cause the material to swell and lead to inaccuracies.
- ▶ Avoid contact with retraction solutions as retraction solution residue can increase the setting time in some cases. Remove retraction solutions by spraying and rinsing thoroughly with water.
- ▶ Do not combine the material with condensation-curing silicone or polyether impression materials.
- ▶ Do not allow the impression material to come into contact with latex gloves as this inhibits the setting process. In the event of prior contact with latex gloves, wash hands carefully. The use of PVC or PE-based gloves is recommended.

Using the Automix cartridge

Inserting the cartridge

1. Push the lever [Fig. 1, pos. A] at the back of the Automix-Dispenser and pull back the slide [Fig. 1, pos. B] completely.
2. Lift up the plastic lever [Fig. 1, pos. C], insert the cartridge and lock it using the plastic lever [Fig. 1, pos. C].

Note: Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

Attaching a mixing tip

1. Remove the cartridge cap [Fig. 1, pos. D] or used mixing tip after turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [Fig. 1, pos. E].

Note: In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available in refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Applying material

- ▶ When using a cartridge for the first time, squeeze out approx. a pea-sized amount of material and discard it.
- ▶ Mix the material in the mixing tip using the lever [Fig. 1, pos. F] of the Automix-Dispenser and apply directly.

Note: After application, leave the used mixing tip on the cartridge as a seal!

Removing the cartridge

1. Push up the lever [Fig. 1, pos. A] at the back of the Automix-Dispenser and pull back the slide [Fig. 1, pos. B] completely.
2. Lift up the plastic lever [Fig. 1, pos. C] and remove the cartridge.

Recommended use

Creating a bite registration model

1. Apply the bite registration material directly to the occlusal surfaces. Coat prepared surfaces first where appropriate.

Note: It is usually sufficient to apply the material to the teeth of the upper or lower dental arch. In the area of the front teeth, it is recommended that the material be applied to both dental arches. A material layer approx. 5 mm thick should be sufficient.

2. Allow the patient to adopt the desired occlusion.
3. After 1:30 minutes, carefully remove the finished bite registration model from the mouth.

Processing the bite registration model

Note: The material has a high final hardness and can be processed using a rotating instrument (e.g. grinder). Immediately after removal, excess can also be removed using a scalpel.

1. Remove excess material that extends beyond the occlusal surfaces of the impression.
2. Remove shreds of material that extend into the proximal areas.

Disinfection of the bite registration model

- ▶ Disinfect bite registration models using standard disinfection solutions.

Storing the bite registration model

Special storage conditions are not required. The bite registration model can be stored for long periods of time.

Optical scanning

The quality of the optical scanning results may vary depending on the impression scanner. For optimum results, DMG recommends the use of the D700 dental scanner from 3Shape. Observe the manufacturer's instructions for the impression scanner used.

Disinfection of the Automix-Dispenser

The Automix-Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser is also autoclavable.

Technical data

Total working time*	≤ 0:30 min
Setting time in the mouth	≥ 1:30 min
Shore-A hardness (1 hour)	≥ 93

*The specified time applies at a room temperature of 23 °C and a normal relative humidity of 50%. Higher temperatures shorten this time while lower temperatures increase it.

Storage and shelf life

- ▶ Store in a dry place at room temperature (15–25 °C / 59 –77 °F).
- ▶ Do not store in the fridge or freezer.
- ▶ Keep away from direct sunlight.
- ▶ After application, leave the used mixing tip on the cartridge as a seal.

Composition

Vinyl polysiloxanes, hydrogen polysiloxanes, fillers, pigments, additives, platinum catalyst

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