# StatusBlue

DMG Chemisch-Pharmazeutische Fabrik GmbH  
 Elbgaustraße 248 · 22547 Hamburg · www.dmg-dental.com  
 904490/#1\_2012-03

## Handling Cartridges

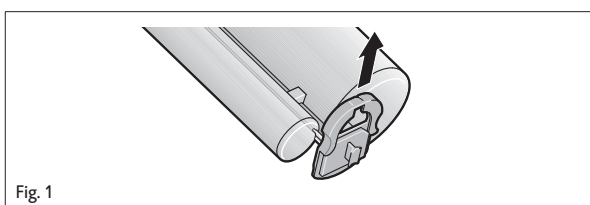
Fig. 1

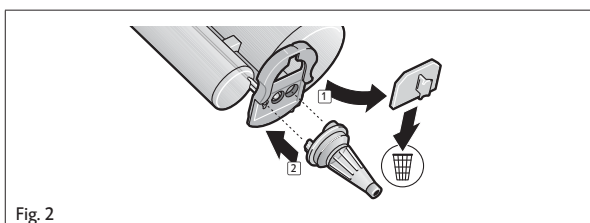
Fig. 2

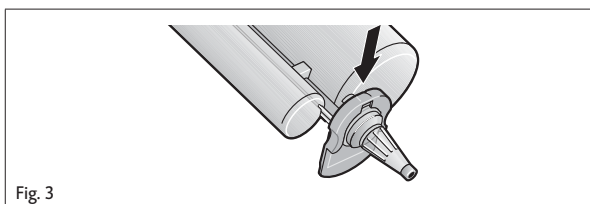
Fig. 3

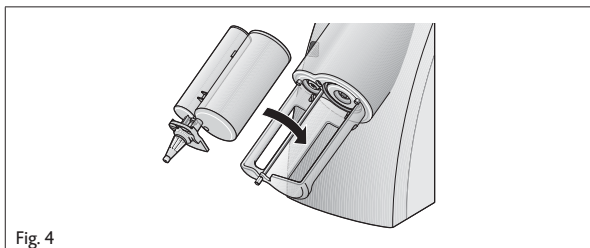
Fig. 4

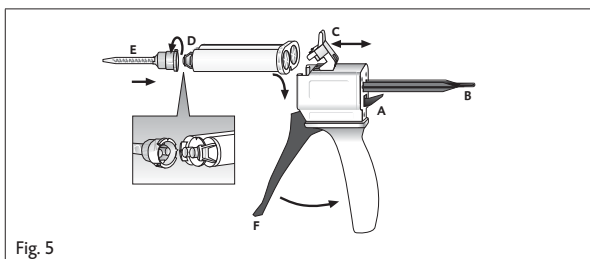
Fig. 5

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## Deutsch

### Produktbeschreibung

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Polysiloxane, Siliciumdioxid, Pigmente, Additive, Platinkatalysator

#### Klassifikation

ISO 4823:2000, Type 2: Medium-bodied consistency

#### Handelsformen

StatusBlue-MixStar	
1 MixStar-Kartusche à 380 ml Paste 10 MixStar-Tips	REF 909693 In den USA REF 999693
5 MixStar-Kartuschen à 380 ml Paste 50 MixStar-Tips	REF 909694
StatusBlue-Automix	
2 Automix-Kartuschen à 50 ml Paste 4 Automix-Tips	REF 909790
8 Automix-Kartuschen à 50 ml Paste 16 Automix-Tips	REF 909791 In den USA REF 999827

## Instructions for Use

## English

#### Product description

StatusBlue is a situation impression material based on addition curing silicones. StatusBlue is available in the MixStar cartridge which is suitable for use in the DMG MixStar-eMotion and other commercially available mixing machines, and also in the 4:1 Automix safety cartridge.

#### Indications

- Impressions for the production of temporaries
- Situation impressions
- Impressions of opposing jaw
- Impressions for making vacuum-formed stents

#### Basic safety instructions

- For dental use only!
- Keep out of reach of children!

#### Interactions

- Never allow the impression to come into contact with organic solvents or liquids containing solvents as this may result in a swelling of the material and may therefore lead to inaccuracies in the cast.
- Avoid contact with retraction solutions, because in some instances residues of retraction solutions can increase the setting time.
- Avoid contact between the impression material and latex gloves because these prevent the setting process. In the event of prior contact between the hands and latex gloves, wash hands thoroughly. It is recommended that PVC or PE based gloves be used.
- Do not mix StatusBlue with condensation curing silicone or polyether impression materials.

## Using the MixStar cartridge in the DMG MixStar-eMotion

- ▶ Follow the instructions for use for the “MixStar-eMotion”!
- 1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge (Fig. 1).
- 2. Remove the cap from the cartridge and discard. Attach a new mixing tip (Fig. 2).
- 3. Secure the mixing tip by sliding the locking mechanism into the locked position (Fig. 3).
- 4. Insert the cartridge into the MixStar-eMotion (Fig. 4). A brief acoustic signal confirms the correct insertion of the cartridge.

**Please note:** Prepare fresh cartridges for use by extruding and discarding the first 3 cm of impression material.

**Please note:** After the application, leave the used tip on the cartridge to cap unused material.

## Use in other mixing machines

- ▶ Follow the instructions of the mixing unit you are using!
- ▶ First insert the cartridge into the mixing unit without the mixing tip. Otherwise appropriate mixing of the material cannot be guaranteed!
- ▶ In this case proceed as described in the section “Changing the mixing tip”.

## Changing the mixing tip

- ▶ Follow the instructions of the mixing unit you are using!

**Please note:** Attach a new mixing tip before each mixing process.

1. Slide the locking mechanism upwards.
2. Remove the old mixing tip and any hardened material blocking the cartridge openings.
3. Place the mixing tip securely on the respective cartridge opening.
4. Lock the mixing tip in place with the locking mechanism.

## Using the Automix cartridge

### Insert cartridge

1. Push up the lever [Fig. 5, Pos. A] at the back of the Automix dispenser and pull back the slide [Fig. 5, Pos. B] completely.
2. Lift up the plastic lever [Fig. 5, Pos. C], insert the cartridge and lock it using the plastic lever [Fig. 5, Pos. C].

**Please note:** Make sure that the notches on the cartridge line up with those on the Automix dispenser.

### Attach mixing tip

1. Remove the cartridge cap [Fig. 5, Pos. D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [Fig. 5, Pos. E].

**Please note:** In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

**Please note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

### Applying the material

- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [Fig. 5, Pos. F] of the Automix dispenser and it is then applied directly.

**Please note:** After the application, leave the used mixing tip on the cartridge to cap unused material!

### Remove cartridge

1. Push up the lever [Fig. 5, Pos. A] at the back of the Automix dispenser and pull back the slide [Fig. 5, Pos. B] completely.
2. Lift up the plastic lever [Fig. 5, Pos. C] and remove the cartridge.

## Instructions for use

- ▶ StatusBlue can be used in all common impression trays. In order to improve the adhesion to metal or plastic trays, a tray adhesive can be applied.
- ▶ For the individual times, refer to the “Technical data” chapter.

## Recommended use

1. Fill the prepared tray with the material.

**Please note:** To avoid air bubbles, the end of the mixing tip should stay immersed in the impression material throughout the application process.

2. Insert the tray into the mouth no more than 1:15 min. after the start of mixing and maintain this position for at least 1:45 min.
3. Remove the tray from the mouth once the setting time has ended.

## Removal of the impression

- ▶ Remove the impression parallel to the longitudinal axis of the prepared teeth.
- ▶ After removal, proceed as described in the section “Cleaning and disinfection of the final impression”.

## Cleaning and disinfection of the final impression

- ▶ Only disinfect the final impression using dipping solutions. DMG recommends the use of a 2% glutaraldehyde solution.
- ▶ Please ensure that the disinfectant used is indicated for use with addition curing silicone impression material.
- ▶ Follow the instructions of the disinfectant solution’s manufacturer precisely!
- ▶ Leave the final impression to dry and store until casting, see “Storage of the impression”.
- ▶ For more information, refer to the comments of the DGZMK. Comments of the DGZMK 3/93 V 1.0; [www.dgzmk.de](http://www.dgzmk.de)
- ▶ ADA recommends following Council on Dental Materials, Instruments and Equipment.
- ▶ Disinfection of impressions. JADA 1991;122(3):110; [www.ada.org](http://www.ada.org)

## Casting and galvanization

- ▶ The impression should be cast 10 minutes at the earliest after removal from the mouth.

It is recommended to use special purpose hard plaster (ISO 6873; Type 3, e.g. Moldano, Bayer) for the cast and an ultra hard dental plaster for the forms (ISO 873, Type 4; e.g. Fujirock II, GC). The impression can be copper-plated or silver-plated.

## Storage of the impression

- ▶ Store impressions at room temperature (15 – 25 °C/59 – 77 °F) in a dry place.

The impression is stable and can be cast even after months without loss of accuracy.

## Production of temporaries

All temporary crown and bridge materials on the market may be used. The bonding of these materials takes place somewhat faster than when alginate impressions are used.

## Disinfection of the Automix dispenser

The Automix dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute). The Automix dispenser is also autoclavable.

## Technical data

Working time*	1:15 min
Recommended time in mouth	1:45 min
Dimensional change after 24 h	≤ 0.4%
Compression set	≤ 0.7%
Strain in compression	≈ 7.0%

\* The times apply at a room temperature of 23 °C and a normal relative air humidity of 50%. Higher temperatures shorten, lower temperatures increase this time.

## Storage and shelf-life

- ▶ Store at room temperature (15 – 25 °C/59 – 77 °F) in a dry place.
- ▶ Do not store in the fridge or freezer.
- ▶ Protect against exposure to direct sunlight.
- ▶ After the application, leave the used tip on the cartridge to cap unused material.
- ▶ Do not use after the expiration date.

## Composition

Polysiloxanes, silicium dioxide, pigments, additives, platinum catalyst

## Classification

ISO 4823:2000, type 2: Medium-bodied consistency

## Available product sizes

StatusBlue – MixStar	
1 MixStar cartridge @ 380 ml paste 10 MixStar tips	REF 909693 In the USA: REF 999693
5 MixStar cartridges @ 380 ml paste 50 MixStar tips	REF 909694

StatusBlue-Automix	
2 Automix cartridges @ 50 ml paste 4 Automix tips	REF 909790
8 Automix cartridges @ 50 ml paste 16 Automix tips	REF 909791 In the USA: REF 999827

Federal law (USA) restricts this device to sale by, or on the order of a dentist, or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

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